## Brugsanvisning Käyttöohjeet Bruksanvisning Productinformatie Οδηγίες Χρήσεως Kullanım kılavuzu Инструкция Instrukcja stosowania

## Instructions for Use Gebrauchsinformation Mode d'emploi Istruzioni d'uso Instruções de Uso Bruksanvisning

- Dualhärdende provisorisk cement på plastbasis
- Kaksoskovettainen resinipohjainen väliaikaissementti
- Dualherdende provisorisk sement på kompositbasis
- Duala uithardend, tijdelijk bevestigingscement op composietbasis
- Σύνθετη προσωρινή κολλητική πολυμερισμού
- Çift sertleşen rezin tabanlı geçici siman
- Временный фиксирующий цемент двойного отверждения на композитной основе
- Cement tymczasowy na bazie materiału kompozytowego, o podwójnym mechanizmie polimeryzacji

- Dual-curing resin based temporary cement
- Dualhärtender temporärer Befestigungszement auf Compositebasis
- Colle provisoire duale
- Cimento composito provvisorio a polimerizzazione duale
- Cemento provisional en base a composites de polimerización dual
- Cimento temporário, baseado em resina, com polimerização dual
- Dualhärdande resinbaserat temporärt cement

For dental use only.  
Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Complies with:  
ISO 4049, EN ISO 4049

CE 0123

ivoclar  
vivadent®  
clinical

## Instructions for Use

### Product Description

Telio CS Link is a dual-curing (light- and self-curing) temporary composite cement to be used for the aesthetic temporary cementation of provisional crowns, bridges, inlays, onlays and veneers. Telio CS Link is compatible with temporary restorations fabricated with resin-based crown and bridge materials (e.g. Telio CS C&B) as well as with restorations made of Telio CAD and/or Telio Lab.

### Shades

Transparent, Yellow (A3)

### Composition

Telio CS Link is composed of bismethacrylates (56 wt.%) and fillers (43 wt.%). Initiators, stabilizers and pigments are additional ingredients.

### Indications

- Temporary cementation of provisional restorations.
- The product has been designed to remain in the mouth for a maximum of 6 weeks.

### Contraindication

The product should not be used in patients with a known allergy to any of its ingredients.

### Side effects

In rare cases, components of Telio CS Link may lead to a sensitizing reaction in susceptible individuals. In these cases, the use of the product should be discontinued.

### Interactions

The eugenol-free formula makes Telio CS Link an ideal temporary cement even if adhesive luting techniques are subsequently employed. Eugenol-containing cements are contraindicated in these cases, as they may affect the

polymerization of the luting composite used for the permanent cementation of the final restoration.

To prevent Telio CS Link from permanently adhering to composite-based core build-up material, freshly placed and prepared composite cores should be coated with a thin layer of separator. Water-soluble glycerine gel (e.g. Liquid Strip) may be used for this purpose. Following cavity preparation, no dentin adhesives or protective varnishes should be used prior to the application of Telio CS Link.

### Application

- 1) Remove the cap of the double-push syringe by turning it 1/4 turn counter clockwise (**discard the cap, do not re-use it!**) and replace it with a mixing tip. If the double-push syringe has been used before, replace the used mixing tip with a new one just before dispensing Telio CS Link.
- 2) When attaching the mixing tip to the syringe, make sure that the guidance of the double-push syringe is aligned with that of the mixing tip. Push the tip completely down until the notch on the mixing tip is aligned with that of the double-push syringe. Secure the mixing tip in place by gripping the coloured base and turning it 1/4 turn clockwise. The double-push syringe contains pre-dosed amounts of Telio CS Link base and catalyst which are automatically mixed and dispensed when the two components are extruded. Thus the material may be directly applied to the provisional restoration.
- 3) Spread a layer of Telio CS Link over the dry inner surfaces of the provisional restoration and/or the prepared, cleaned and dried tooth surfaces. The working time is approx. 2.5 to 3 minutes at a room temperature of 23 °C/73 °F. Incorporation of Telio CAD temporary restorations: Abrade the inside surfaces of the restoration by abrasive (with Al<sub>2</sub>O<sub>3</sub>, 100 µm grit at 1-2 bar/15-29 psi) or with a coarse diamond bur.
- 4) Seating of the provisional restoration and removal of excess cement  
Seat the provisional restoration with firm but light pressure. Excess cement can be removed using different methods.

- 4a) Removal of excess with additional light-activation (quarter technique)  
Light-cure excess material with the polymerization light (approx. 650 mW/cm<sup>2</sup>, e.g. bluephase, LOP mode,) for 2-4 seconds per quarter surface (mesio-oral, disto-oral, mesio-buccal, disto-buccal) at a distance of approx. 0-10mm. Following this, the excess material can be easily removed with a scaler as it features a viscoplastic consistency. Subsequently, light-cure all margins again for 10 seconds (light intensity > 1,000 mW/cm<sup>2</sup>, e.g. bluephase, HIP mode).
- 4b) Removal of excess in the unpolymerized state  
Remove uncured excess material immediately after seating using a microbrush / brush / foam pellet / dental floss or scaler. Subsequently, wait until the self-curing process is finished (approx. 3 minutes) or accelerate the polymerization process by light-curing the cement layer with a curing light (light intensity > 1000 mW/cm<sup>2</sup>, e.g. bluephase®, HIP mode) for 10 seconds per aspect.
- 4c) Removal of excess in the polymerized state  
Approx. 3 minutes after having seated the restoration, remove excess material carefully with a scaler or other instruments.

**Leave the used mixing tip on the syringe.  
It will serve as a seal until the next use!**

### Special notes

- To ensure an easy clean-up and complete removal of excess, we recommend holding back the lip or cheek surfaces during the setting process, to prevent the excess material from spreading out into a thin layer.
- During the self-curing process, the patient should not exert any pressure on the provisional restoration.

### Working time

Approx. 2.5 to 3 minutes at a room temperature of 23 °C / 73 °F.

### Self-curing time

Four minutes at a temperature of 37 °C / 98 °F.  
Exposure to light will accelerate the curing process.

## Note

- Extruding material without the mixing tip in place may result in a cross-contamination of the Telio CS Link components.
- Like all other composites, Telio CS Link is subject to oxygen inhibition. Therefore, the behaviour of the cement when mixed on a mixing pad outside the mouth cannot be taken as a reference for the intraoral setting behaviour.

## Precautions

- Prevent any contact of uncured material with the skin/mucous membrane and eyes. Unpolymerized Telio CS Link may cause slight irritation and in very rare cases, sensitization may occur if the skin is exposed repeatedly.
- Commercially available medical gloves do not offer protection against the sensitizing effect of methacrylates.

## Storage and shelf life

- Shelf life: see expiration date on label/packaging
- Do not use Telio CS Link after the indicated date of expiration.
- Storage temperature: 2 to 8 °C / 36 to 46 °F
- Only use Telio CS Link at room temperature (cooled material is more viscous and exhibits a slower setting behaviour).

## Keep out of the reach of children!

### For use in dentistry only!

**Date information prepared:** 06/2010, Rev. 1

## Manufacturer

Voclar Vivadent AG, FL-9494 Schaan/ Liechtenstein

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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## Ivoclar Vivadent UK Limited
## Ivoclar Vivadent, Inc.