# bluephase®

- **Proper handling** of lithium polymer batteries (LiPo)
- Der **richtige Umgang** mit Lithium-Polymer-Akkus (LiPo)
- **Conseils d'entretien** des batteries lithium polymère (LiPo)
- Come **utilizzare correttamente** le batterie ai polimeri di litio (LiPo)
- **Manejo correcto** de baterías de polímero de litio (LiPo)
- **O manuseamento correcto** de baterias de lítio polímero (LiPo)
- **Korrekt hantering** av litium-polymerbatterier (LiPo)
- **Korrekt håndtering** af lithium-polymer-batterier (LiPo)
- Litiumpolymeeriakkujen (LiPo) **oikea käsittely**
- **Riktig behandling** av litium-polymer-batterier (LiPo)
- **Correct omgaan** met lithiumpolymeer-batterijen (lipo-batterijen)
- **Κατάλληλος χειρισμός** των μπαταριών λιθίου-πολυμερών (LiPo)
- Lityum polimer pillerin (LiPo) **doğru kullanımı**

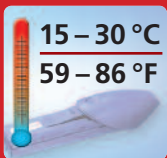
1

2

3

4

5

6

7

## **1 PROPER CHARGING**

For as long a life cycle of the battery as possible, return the light to the charging base as often as possible – e.g. after each patient.

## **2 CLEANING**

Battery contacts should be cleaned regularly to ensure that they are capable of conducting and therefore charging properly – e.g. by means of mandatory wipe disinfection of the light before each use.

## **3 STORAGE / PRACTICE HOLIDAYS**

In order to prevent premature wear, always charge the battery and do not store batteries for longer than 6 months.

## **4 PROTECT FROM EXTREME TEMPERATURES**

For full operational reliability, always use (and store) the battery at room temperature or between 15 °C (59 °F) and 30 °C (86 °F).

## **5 REGULAR REPLACEMENT**

Since the battery is a consumable, it has to be replaced if the performance drops considerably or after its typical life cycle has expired after approx. 2½ years.

## **6 DO NOT PURCHASE BATTERIES IN ADVANCE**

In order to avoid irrevocable deep discharge if the battery is not used, a new battery is best purchased only once the old LiPo battery needs to be replaced.

## **7 NO DISPLAY DURING CHARGING**

Check if the handpiece has been correctly placed into the charging base. Clean battery contacts. Information: "Charging Battery" briefly appears in the display before the display returns to the stand-by mode.

If cleaning the battery contacts does not improve the results, remove the battery from the apparatus. Charge the battery separately from the handpiece in the charging base for approx. 10 minutes. If the error remains, please contact your dealer or your local Service Center.

## **8 SAVETY NOTE**

Please note that the lithium polymer battery may react with explosion, fire, and smoke development in case of inappropriate handling and mechanical damage.

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## **Ivoclar Vivadent Limited**
## **Ivoclar Vivadent, Inc.**