# Fluor Protector

English

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## Instructions for Use

### Description:

Fluor Protector is a protective varnish for desensitization and caries prophylaxis. It contains fluoride.

### Composition:

1g Fluor Protector contains:

Bis {4 - [2- (difluorhydroxysilyl) ethyl ] – 2 – methoxycyclohexyl } [N,N- (trimethylhexane – 1,6 – diyl) dicarbamate] (9 mg) (fluorsilane).

This corresponds to 1mg fluoride.

Auxiliary substances: Acetylacetate, isopentyl propionate, polyurea-forming varnish

### Indication:

- Treatment of hypersensitive cervicals
- Improvement of enamel resistance
- Long-term caries prophylaxis

### Contraindication:

If patients are known to be allergic to any of the ingredients in Fluor Protector, the material should not be used.

### Side effects:

A slight burning sensation may occur if Fluor Protector comes in contact with gingival tissue. Hypersensitivity reactions (allergic reactions) in individual cases cannot be excluded. According to today's standard of knowledge, swallowed varnish particles are harmless.

### Application:

Fluor Protector is professionally applied by dentists, dental hygienists or dental assistants with extended duties to give prophylactic treatment and can be used for patients of all age groups. Generally, Fluor Protector is applied every six months. Within the framework of intensive treatment, Fluor Protector can be applied at shorter intervals.

For dental use only.  
Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein  
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

**CE 0482**

**ivoclar**  
**vivadent**<sup>®</sup>  
*clinical*

**Ampoule:** The contents are sufficient for 2–3 sets of teeth.

**VivAmpoule:** The contents are sufficient for 1 set of teeth.

**Step-by-step procedure:**

1. Thoroughly clean the tooth surfaces.
2. Create a dry field with cotton rolls and air syringe.
3. Open VivAmpoule with the integrated opening aid or place ampoule in the plastic base and open it with the enclosed breaker.
4. Apply a thin layer of Fluor Protector using a suitable single-use applicator, e.g. Vivabrush or a brush. Use dental floss to apply the material in proximal areas.
5. Evenly disperse and dry the varnish, optionally with an air syringe.
6. Remove cotton rolls after 1 minute.
7. Do not ask the patient to rinse the mouth.

**Tips for patients:**

After the application of Fluor Protector, patients should neither eat nor brush their teeth for 45 minutes.

**Recommendation:**

Fluor Protector should be used up quickly after opening the VivAmpoule or ampoule, since the material tends to evaporate.

**Note:**

Highly flammable.

The product contains ethyl acetate.

Keep away from sources of ignition.

- **Remove spilled varnish immediately to prevent stains.**

**The varnish should not come in contact with water or aqueous cleaning solutions. Up to 12 hours after spilling, acetone or ethyl acetate can be used to remove the varnish.**

**Note: Check the surfaces of practice equipment for solvent-resistance first. After longer periods of time, the spilled varnish can only be removed mechanically.**

**Storage:**

- Store Fluor Protector at 2–28 °C (36–82 °F)
- Shelf life: see date of expiration (if original packaging remains unopened)
- Do not use the material after the indicated date of expiration

**Keep the material out of the reach of children!  
For dental use only.**

**Date information prepared:** 03/2011, Rev. 3

**Manufacturer:**

Ivoclar Vivadent AG

FL-9494 Schaan

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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**Ivoclar Vivadent UK Limited**
**Ivoclar Vivadent, Inc.**