Brugsanvisning

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Bruksanvisning

Productinformatie

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633299/0410/W/E3/G

For dental use only.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein
Ivoclar Vivadent AG, FL-9494 Schaan /
Liechtenstein

ivoclar
vivadent®
clinical

Instructions for Use

Description

Telio Add-On Flow is a light-curing, flowable composite for adjustments and repairs as well as for the customization of the emergence profiles of temporary Telio restorations.

Shade

Telio Add-On Flow is available in one universal shade.

Composition

Bis-GMA, urethane dimethacrylate and triethylene glycol dimethacrylate, fillers, initiators, stabilizers and pigments. Total inorganic filler content: 30%vol., filler particle size: 0.04 – 0.20 µm.

Indications

For Telio CS C&B, Telio CAD and Telio Lab restorations.

- Supplementing of contact points
- Filling of air bubbles / inhomogeneities
- Repairs
- Occlusal adjustments / occlusal build-up
- Customization of the emergence profile
- Relining of and/or supplementing of pontics

Contraindications

- Use for permanent restorations.
- If a dry intraoral working field cannot be established or the stipulated application technique cannot be used.
- If a patient is known to be allergic to any of the ingredients of Telio Add-On Flow.
- Application on C&B materials other than Telio CAD, Telio Lab and Telio CS restorations.

Side effects

In rare cases, components of Telio Add-On Flow may cause a sensitizing reaction. Telio Add-On Flow should not be used in such cases.

Important processing notes

The following processing errors must be avoided in order to ensure successful results with Telio Add-On Flow:

- Insufficient surface conditioning

- Combination with materials that are not approved and/or recommended
- Insufficient polymerization

Interactions

- Materials containing eugenol/clove oil inhibit the polymerization of composites. Such materials must not be used in conjunction with Telio Add-On Flow.
- Discolouration may occur in combination with cationic mouthwashes, plaque disclosing agents and chlorhexidine.

Application in the dental practice

• Direct addition of material during the fabrication of Telio CS C&B temporaries

Newly fabricated Telio CS C&B temporaries can be supplemented or repaired with fresh material following the removal of the inhibition layer or, in general, after having ground the area to be adjusted. Telio Add-On Flow is directly applied to the prepared area and cured according to the table below. In areas where retention is limited, the bond between the different layers may be enhanced by pre-treating the surfaces as described in section "Repair / adjustment of Telio CS C&B, Telio CAD, Telio Lab".

Finally, polish the surfaces using silicon carbide rubber polishers (e.g. Astropol®).

• Repair / adjustment of Telio CS C&B, Telio CAD, Telio Lab temporaries

Roughen the area to be supplemented or repaired using coarse diamonds or sandblast (Al_2O_3 , 100 μ m, 1–2 bar/15–29 psi). Thoroughly rinse with water and dry with oil-free compressed air. Then wet the surfaces to be bonded to externally with Telio Activator. Brush Telio Activator into the entire surface for at least 30 seconds using an application brush in order to ensure an even distribution and quick penetration of the Activator. Subsequently, allow the Activator to react for another 30–60 seconds (total reaction time 1–2 minutes). Now apply the Heliobond bonding agent, thinly disperse it with blown air and polymerize for 10 seconds (650 mW/cm², e.g. bluephase® using LOP).

Then apply Telio Add-On Flow and cure according to the table below.

Finally, polish the surfaces using silicon carbide rubber polishers (e.g. Astropol).

As an alternative, the conditioned areas may also be repaired and supplemented with a light-curing Ivoclar Vivadent composite (e.g. Tetric EvoFlow®) following preparation.

• Polymerization times (Telio Add-On Flow)

Polymerization device	Manufacturer	Output mW/cm ²	Polymerization time
bluephase C8	Ivoclar Vivadent	800 mW/cm ²	20 s
bluephase	Ivoclar Vivadent	1200 mW/cm ²	15 s
bluephase 20i	Ivoclar Vivadent	2000 mW/cm ²	2x 5 s

• Finishing/polishing

After polymerization, remove excess material with suitable finishers (e.g. Astropol F) or fine diamonds. Check the occlusion and articulation and adjust by grinding. Use silicon carbide rubber polishers (e.g. Astropol P / Astropol HP) to polish the restoration to a high gloss.

Application in the dental laboratory

• Repair / adjustment of Telio CS C&B and Telio Lab temporaries

Roughen the area to be supplemented by grinding or sandblast (Al_2O_3 , 100 μ m, 1–2 bar/15–29 psi). Thoroughly clean with steam and dry with oil-free compressed air. After that, condition the restoration with Telio Lab Cold Liquid or Telio Activator. For that purpose, distribute Telio Activator evenly and generously on the conditioned surface and allow it to react for at least 2 minutes to a maximum of 4 minutes. Apply SR Compositiv to the activated surface according to the respective Instructions for Use and polymerize. Apply Telio Add-On Flow to the conditioned surface, contour to achieve the desired shape and polymerize.

The temporaries can also be additionally supplemented with Telio Lab LC materials (See Telio Lab LC Instructions for Use).

• Polymerization times for Telio Add-On Flow

The respective light-curing Telio components are pre-polymerized to ensure that they stay in place and to be able to apply a further layer. However, final polymerization of the entire restoration is always required.

Polymerization device	Manufacturer	Pre-polymerization	Final polymerization
Quick	Ivoclar Vivadent	40 s (SR Compositiv 60 s)	—
Lumamat® 100 (Targis Power Upgrade)	Ivoclar Vivadent	Exclusively use Quick for pre-polymerization	11 min (program P2)
Spectramat®	Ivoclar Vivadent	5 min (Telio Stains (2.5 min))	5 min
Visio Beta ¹	3M ESPE	7 min, of which 10 s vacuum (program depends on type of device)	15 min with vacuum (program depends on type of device)
UniXS ¹	Heraeus	90 s	2x 180 s
Solidilite EX ¹	Shofu	1 min	5 min

• Finishing / polishing

Remove excess and finish the surface using suitable grinding instruments. Use fine-grit diamonds or fine burs. Work with suitable speed and little pressure. Smooth out the surface with rubber polishers and silicone polishing wheels. Intermediate polishing is performed with goat hair brushes in conjunction with the Universal Polishing Paste and/or fine pumice. Polish the restoration to a high gloss using cotton or leather buffing wheels as well as the Universal Polishing Paste.

Note

The surfaces of polished Telio Add-On Flow temporaries that need to be supplemented with additional Telio Add-On Flow must be roughened by blasting with Al₂O₃ prior to applying fresh material.

Warning

- Avoid contact of uncured Telio Add-On Flow with the skin / mucous membrane and eyes. Uncured Telio Add-On Flow may cause slight irritation and may lead to a sensitization against methacrylates.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage

- Storage temperature 2-28 °C (36-82 °F).
- Shelf life: see expiry date on label/packaging.

- Do not use Telio Add-On Flow after the indicated expiry date.
- Syringes should be closed immediately after use. Exposure to light causes premature polymerization.

Keep out of the reach of children!

For use in dentistry only!

Date information prepared: 02/2010, Rev. 0

Manufacturer

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

The product has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes.

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¹ not a registered trademark of Ivoclar Vivadent AG
Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.