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Bruksanvisning

Productinformatie

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For dental use only!

Caution: U.S. Federal law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein

Ivoclar Vivadent AG, FL-9494 Schaan / Liechtenstein

CE 0123

573127/1010/WE3/H

ivoclar
vivadent®
clinical

Instructions for Use

Description

Compoglass® F is a compomer-based, light-curing radiopaque single-component restorative material. Compoglass F cures with light in the wavelength range of 400–500 nm (blue light). Compoglass F combines the advantages of glass ionomer cements with those of light-curing composite materials.

Shades

Compoglass F is available in the following shades (Chromascop/A-D Shades):

105/ –, 110/ A1, 140/ A2, 210/ A3, 230/ A3,5, 310/ B3, 340/ A4, 410/ D3, 510/ C3, 540/ –

Composition

The monomer matrix consists of dimethacrylates (19.3 wt.%). The inorganic fillers are ytterbium trifluoride, Ba-Al-fluorosilicate glass, and spheroid mixed oxide (80.5 wt.%). Additional contents are catalysts, stabilizers and pigments (0.2 wt.%).

The total content of inorganic fillers is 57.5 vol.% or 80.5 wt.%. The particle size is between 0.2 µm and 3.0 µm.

Indication

- Restorations in deciduous teeth
- Class V restorations (cervical caries, root erosion, wedge-shaped defects)
- Class III restorations
- Intermediate Class I and II restorations

Contraindication

Compoglass F restorations are contraindicated if the patient is known to be allergic to any of the ingredients in Compoglass F, or if the stipulated application technique cannot be used.

Compoglass F is contraindicated for direct or indirect pulp capping, as well as stress-bearing permanent restorations in secondary dentition.

Side effects

In rare cases, components of Compoglass F may cause sensitivity. In these cases, the material should no longer be applied. To avoid irritation of the pulp, areas near the pulp should be covered with a suitable pulp/dentin protecting material (apply a calcium hydroxide preparation in areas close to the pulp and subsequently cover with a suitable liner).

Interactions

Materials containing eugenol/clove oil inhibit the polymerization of composites. Therefore, the use of zinc oxide eugenol cements in combination with Compoglass F should be avoided. Contact with cationic mouthwashes, plaque disclosing agents, and chlorhexidine may lead to discoloration.

Application

1. Shade determination

Clean the teeth prior to shade determination. Select the shade while the teeth are still moist.

2. Isolation

Isolate with cotton rolls or best with rubber dam (e.g. OpraDam® Plus), depending on the clinical situation.

3. Cavity preparation

Cavity preparation is done according to the principles of the adhesive technique, i.e. preserving tooth substance. Do not prepare sharp edges or additional undercuts in caries-free regions. The cavity geometry is primarily determined by the extent of the caries or the size of the previous restoration.

Bevel the enamel margins in the anterior region and remove sharp edges in the posterior region. Do not prepare caries-free cervical defects. Instead, clean with pumice or a suitable cleansing paste (e.g. Proxyl RDA 36) and a rubber cup or rotary brush. Remove a thin layer of sclerous cervical dentin, using a rotary instrument (diamond finisher, coarse-grained disc). Then, remove all residue in the cavity preparation with water spray. Avoid contamination with saliva following cleaning.

4. Pulp protection / Bases and liners

Only cover very deep areas close to the pulp with a calcium hydroxide material and subsequently apply a pressure resistant cement (eg a glass ionomer cement such as Vivaglass® Liner). Do not cover the remaining cavity walls so that they can be used to generate a bond with an enamel-dentin adhesive.

5. Placing the matrix / interdental wedge

If the cavity includes proximal areas, use a wrap-around matrix (e.g. the selectively thinned OpraMatrix®) or a sectional matrix and wedge it.

The use of a retraction cord or cervical matrix (e.g. Contour Strip) may be helpful for cervical defects. Rinse the cavity with water spray and subsequently dry with oil-free air. Avoid dehydration of the dentin.

6. Conditioning / Application of the bonding agent

Condition and apply the bonding agent according to the Instructions for Use of the product in use. Ivoclar Vivadent recommends using Syntac® bonding agent (in combination with phosphoric acid etching) or ExiTE® F (in combination with phosphoric acid etching) or the self-etching AdheSE® One F adhesive.

7. Application of Compoglass F

Apply Compoglass F in layers of max. 3 mm (light shades) or 2 mm (dark shades = 340/A4 and 540/-) and adapt with a suitable instrument.

Polymerize each layer as indicated in the table shown below. Hold the light emission window of the curing light as closely as possible to the surface of the restorative material.

When using a metal matrix, additionally polymerize from the buccal or lingual/palatal after removing the matrix.

Program	Unit	bluephase C8	bluephase	bluephase 20i
Turbo Program		–	–	2x5 s
High Power Program		40 s	20 s	20 s
Soft Start Program		40 s	20 s	20 s
Light intensity		Exposure Time		
≥ 500 mW/cm ²		40 s		
≥ 1000 mW/cm ²		20 s		

8. Finishing / Occlusal check / Polishing

After polymerization, remove excess with suitable finishers or fine diamonds. Remove proximal excess with flexible discs, finishing strips, diamond-coated files, diamond or tungsten carbide finishers. Check occlusion and articulation and adapt, if necessary, to remove premature contacts and undesired occlusal paths on the restoration surface. Polish the restoration to a high gloss using silicone polishers (e.g. OpraPol® Next Generation), as well as polishing discs, and strips.

Warning

- Unpolymerized Compoglass F should not come in contact with skin/mucous membrane or eyes. Unpolymerized Compoglass F may have a slight irritating effect and may lead to a sensitization against methacrylates.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage and hygiene

- Do not use Compoglass F after the indicated date of expiration.
- Expiry date: see note on syringe label or package
- Store Compoglass F at 2–28 °C / 36–82 °F
- Once the foil bag has been opened, Compoglass F may be stored for two months. The material must be used within this period of time. This applies for syringes as well as Cavifils.

- Close syringes and Cavifils immediately after use. Exposure to light causes premature polymerization.
-  If Compoglass F is applied from the Cavifil directly in the mouth of the patient, the Cavifil must not be used for more than one patient due to hygienic reasons (prevention of cross-contamination between patients).
- Do not use oxidizing disinfectants to disinfect syringes or Cavifils.

**Keep out of the reach of children!
For dental use only!**

Date information prepared: 10/2010; Rev. 1

Manufacturer:

Ivoclar Vivadent AG
FL-9494 Schaan/Liechtenstein

These materials have been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

**The Material Safety Data Sheet is available online at
www.ivoclarvivadent.com**

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Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.