Description

OptraGate® ExtraSoft is an auxiliary aid for clinical use which allows the working field to be accessed with ease.

It provides retraction of lips and cheeks during dental treatment, offers the clinician increased visibility and accessibility and facilitates moisture control in the oral cavity. The device, which features high three-dimensional flexibility and elasticity, ensures patient comfort and assists in keeping the patient's mouth open. OptraGate is available in sizes Regular, Small as well as Junior for children between 5 and 10 years of age. The modified "ExtraSoft Version" offers a snug, particularly gentle fit in the area of the upper and lower lip frenulum which minimizes intra-oral pressure.

Composition

Thermoplastic elastomer (SEBS) and polypropylene.
OptraGate ExtraSoft is latex-free.

Indication

Suitable for use in diagnostic, preventive and therapeutic dental procedures (in particular restorative procedures, cavity preparation, professional tooth cleaning, diagnostic procedures, scaling & root planing, tooth whitening, impression making).

Contraindications

The use of OptraGate ExtraSoft is contraindicated

- in oral surgical and implantological procedures;
- if the patient is known to be allergic to any of the components of OptraGate ExtraSoft.

Description of the procedure

The elastic material of OptraGate ExtraSoft is stretched by mean of two rings. The thicker ring of the device is positioned in the area of the gingivobuccal fold, while the thinner ring, which features two tabs, remains outside the mouth. The elastic component between the two rings thus embraces the lips of the patient and provides retraction due to the restoring force of the rings (see Figure 1).

Fig 1: OptraGate ExtraSoft in place with the tabs pointing downward. It does not interfere with the patient's occlusion.

1. Select the appropriate sized OptraGate ExtraSoft. The size (R for regular, S for small or J for Junior) is printed on the package as well as on the left tab of OptraGate ExtraSoft. To ensure optimum function, we recommend inserting the largest size that fits in the patient's mouth.

OptraGate®

**ExtraSoft
Version**

OptraGate®

**ExtraSoft
Version**

EN	Instructions for Use Flexible aid for full mouth access	DA	Brugsanvisning Fleksibelt hjælpemiddel til bedre adgang til mundhulen	SQ	Udhëzime përdorimi Mjet tërheqës elastik për buzët
DE	Gebrauchs-information Flexibles Hilfsmittel für freien Zugang zum Mundraum	FI	Käyttöohjeet Joustava apuväline helpottamaan toimenpiteisiin pääsyä suussa	RO	Instrucțiuni de utilizare Instrument flexibil pentru acces complet în cavitatea bucală
FR	Mode d'emploi Ecarteur flexible permettant un accès facile en bouche	NO	Bruksanvisning Fleksibelt hjælpemiddel for fri tilgang til munnhulen	RU	Инструкция эластичное вспомогательное средство для свободного доступа к полости рта пациента
ES	Istruzioni d'uso Retrattore flessibile per l'accesso al cavo orale	NL	Productinformatie Flexibel hulpmiddel voor vrije toegang tot de mondholte	UK	Інструкція щодо використання Гнучкий засіб для забезпечення максимального доступу до ротової порожнини
IT	Instrucciones de uso Ayuda flexible para el acceso completo a la cavidad bucal	EL	Οδηγίες Χρήσεως Εύκαμπτο βοήθημα για πλήρη πρόσβαση στο στόμα		
		SL	Navodilo za uporabo Fleksibilen pripomoček za poln dostop do ustne votline		
		HR	Uputstvo za uporabu Fleksibilno pomagalo za pristup cijeloj usnoj šupljini		
		CS	Návod k použití Pružná pomůcka umožňující úplný přístup do úst		
		SK	Návod na používanie Pružná pomôcka na úplný prístup do úst		
		HU	Használati utasítás Rugalmas szájterpesz		

- OptraGate ExtraSoft may be inserted from both the 12 o'clock position with the patient lying down, or from the 8 o'clock position with the patient sitting upright (or, the 4 o'clock position if the clinician is left-handed).
- To ease placement, with the tabs pointing downward, the operator should grasp the thicker intraoral ring by reaching through the thinner extraoral ring. The inner ring should be held between the thumb and middle finger and pressed together slightly (see Figure 2).

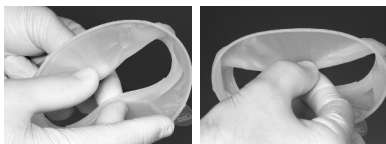
Fig 2: Correct hold on the intraoral ring with thumb and middle finger. The intraoral ring is slightly compressed in the process.

- The slightly compressed intraoral ring is then inserted into the buccal corridor on one side, so that the elastic component embraces the corner of the mouth and the outer ring is still located extraorally (see Figure 3).

Fig 3: The thicker intraoral ring is positioned between the teeth and the corner of the mouth.

- As soon as the OptraGate ExtraSoft is securely in place on one side, the other side is inserted in the same manner by slightly bending it (see Figure 4).

Fig 4: The intraoral ring is positioned behind the two corners of the mouth.

- Subsequently, the intraoral ring is placed behind the lower and upper lips and thus OptraGate ExtraSoft assumes its final, stable position (see Figure 5). The intraoral ring can be inserted with more ease if the patient's mouth is relaxed.

Fig 5: Final positioning behind the lower and upper lip.

If, in individual cases, the intraoral ring should show a tendency to slip out of the gingivobuccal fold if the mouth is closed completely, positioning of the intraoral ring deeper inside the vestibule is usually sufficient to solve the problem. Sometimes, the use of another size may also be a solution.


The securely positioned OptraGate ExtraSoft improves the overall view and accessibility of the working field. Lateral movements of the mandible and opening of the mouth to various degrees may provide additional space and access to the treatment area. OptraGate ExtraSoft may even be left in place when checking occlusion.

- To remove OptraGate ExtraSoft, grasp the extraoral ring in the lower area with the help of a paper towel and lift it slightly so that the intraoral ring is loosened from the lower gingivobuccal fold (Figure 6). The intraoral ring can then be easily removed from the upper gingivobuccal fold and hygienically disposed of using the paper towel.

Fig 6: Removal of OptraGate ExtraSoft

Notes

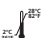-  OptraGate ExtraSoft is not sterile and thus not suitable for use with oral surgical or implantological procedures.
-  OptraGate ExtraSoft is for single use only.
- It cannot be reused. The material is not suitable for sterilization. Cleaning and disinfecting agents may have a detrimental effect on the material and lead to undesirable reactions in the patient.
- When used in tooth whitening procedures, care should be taken that no bleaching material gets caught between OptraGate ExtraSoft and the oral soft tissues to prevent an irritation of the skin and mucous membrane.
-  OptraGate ExtraSoft should be stored at 2-28 °C / 36-82 °F.
-  These Instructions for Use are also available at the website of Ivoclar Vivadent AG (www.ivoclarvivadent.com).

Keep out of the reach of children!
For use in dentistry only!

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www.ivoclarvivadent.com

Ivoclar Vivadent Pty. Ltd.

1 – 5 Overseas Drive, P.O. Box 367, Noble Park, Vic. 3174, Australia

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www.ivoclarvivadent.us

Ivoclar Vivadent Shanghai

Trading Co., Ltd., 2/F Building 1, 881 Wuding Road, Jing An District

200040 Shanghai, China

Tel. +86 21 6032 1657, Fax +86 21 6176 0968

www.ivoclarvivadent.com

Ivoclar Vivadent Marketing Ltd.

Calle 134 No. 7-B-83, Of. 520, Bogotá, Colombia

Tel. +57 1 627 3399, Fax +57 1 633 1663

www.ivoclarvivadent.co

Ivoclar Vivadent SAS

B.P. 118, F-74410 Saint-Jorioz, France

Tel. +33 4 50 88 64 00, Fax +33 4 50 68 91 52

www.ivoclarvivadent.fr

Ivoclar Vivadent GmbH

Dr. Adolf-Schneider-Str. 2, D-73479 Ellwangen, Jagst, Germany

Tel. +49 7961 889 0, Fax +49 7961 6326

www.ivoclarvivadent.de

Wieland Dental + Technik

GmbH & Co. KG, Schwenninger Strasse 13, D-75179 Pforzheim, Germany

Tel. +49 7231 3705 0, Fax +49 7231 3579 59

www.wieland-dental.com

Ivoclar Vivadent Marketing (India) Pvt. Ltd.

503/504 Raheja Plaza, 15 B Shah Industrial Estate

Veera Desai Road, A, ndheri (West), Mumbai, 400 053, India

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www.ivoclarvivadent.com

Ivoclar Vivadent Ltd.

12 Omega St, Rosedale, PO Box 303011 North Harbour

Auckland 0751, New Zealand

Tel. +64 9 914 9999, Fax +64 9 914 9990

www.ivoclarvivadent.co.nz

Ivoclar Vivadent Polska Sp. z o.o.

Al. Jana Pawla II 78, 00-175 Warszawa, Poland

Tel. +48 22 635 5496, Fax +48 22 635 5469

www.ivoclarvivadent.pl

Ivoclar Vivadent Marketing Ltd.

Prospekt Andropova 18 korp. 6/, office 10-06, 115432 Moscow, Russia

Tel. +7 499 418 0300, Fax +7 499 418 0310

www.ivoclarvivadent.ru

Ivoclar Vivadent Marketing Ltd.

Qlaya Main St., Siricon Building No.14, 2nd Floor, Office No. 204

P.O. Box 300146, Riyadh 11372, Saudi Arabia

Tel. +966 11 293 8345, Fax +966 11 293 8344

www.ivoclarvivadent.com

Ivoclar Vivadent Pte. Ltd.

171 Chin Swee Road, #02-01 San Centre, Singapore 169877

Tel. +65 6535 6775, Fax +65 6535 4991

www.ivoclarvivadent.com

Ivoclar Vivadent S.L.U.

C/ Ribera del Loira nº 46, 5ª planta, 28042 Madrid, Spain

Tel. + 34 913 757 820, Fax + 34 913 757 838

www.ivoclarvivadent.es

Ivoclar Vivadent AB

Dalvägen 14, S-169 56 Solna, Sweden

Tel. +46 8 514 939 30, Fax +46 8 514 939 40

www.ivoclarvivadent.se

Ivoclar Vivadent Liaison Office

: Tesvikiye Mahallesi, Sakayik Sokak, Nisantas' Plaza No:38/2

Kat:5 Daire:24, 34021 Sisli – Istanbul, Turkey

Tel. +90 212 343 0802, Fax +90 212 343 0842

www.ivoclarvivadent.com

Ivoclar Vivadent Limited

Ground Floor Compass Building, Feldspar Close, Warrens Business Park,

Enderby, Leicester LE19 4SE, United Kingdom

Tel. +44 116 284 7880, Fax +44 116 284 7881

www.ivoclarvivadent.co.uk

Ivoclar Vivadent, Inc.

175 Pineview Drive, Amherst, N.Y. 14228, USA

Tel. +1 800 533 6825, Fax +1 716 691 2285

www.ivoclarvivadent.us