

Simply universal

Monobond®
Plus

Universal primer



Versatile applications

Correct preparation of the adhesive surfaces of a restoration is an essential part of the adhesive cementation process. Monobond® Plus is a universal primer for conditioning all common restorative materials in everyday practice.




Impressive performance

Used together with the versatile luting composite Variolink® Esthetic DC, Monobond Plus ensures a reliable bond to zirconium oxide and glass-ceramics.^[2]


Durability of the bond established to zirconium oxide and lithium disilicate by different primer/bonding systems^[2]

[Tensile bond strength after immersion in water for 150 days and 37,500 cycles of thermocycling]

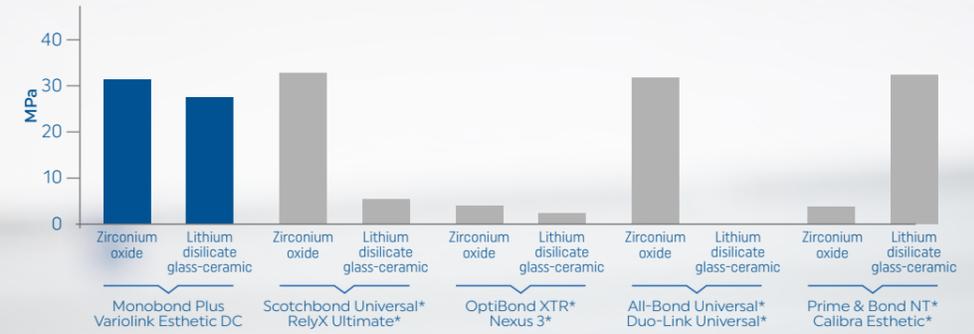

A primer for a wide variety of materials

Monobond Plus contains an innovative combination of functional monomers that promote a strong bond to zirconium oxide, glass-ceramics and metal alloys – comparable to the performance of bonding agents for single materials.^[1]

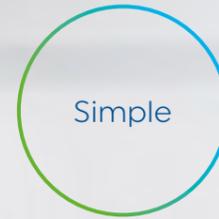


Uniform application protocol with a 60-second contact time

Monobond Plus is used in the same way on all restorative materials: apply, allow to react for 60 seconds and disperse excess.

Tensile bond strength of MultiLink® Automix and Monobond® Plus on various materials^[1]

[Tensile bond strength after immersion in water for 167 days and 37,500 cycles of thermocycling]

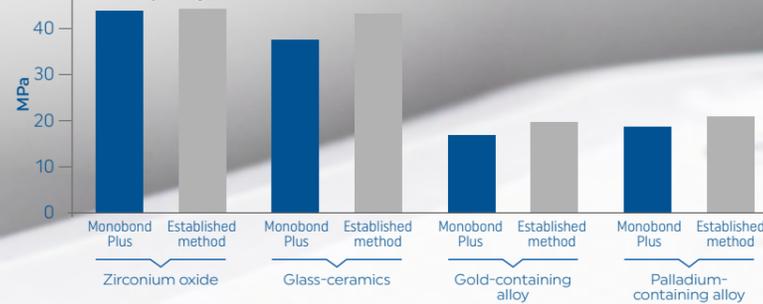

* These brands are not registered trademarks of Ivoclar Vivadent AG.

[1] Azimian et al.; J Adhes Dent 2012; 14: 275–282.

[2] Elsayed et al.; J Adhes Dent 2017; 19: 221–228.

Fixed prosthetics workflow for individual and reliable restorations



Isolate

Establish relative isolation and an easily manageable treatment field with OptraGate



Scan and send

Convenient digital impression taking and immediate transfer of data via Ivoclar Cloud



Select

IPS e.max: all-ceramic materials for nearly all restorative indications



Plan

Select the suitable cementation protocol using the Cementation Navigation System



Condition

Simply condition the bonding surface of a restoration with Monobond Plus



Bond

Generate a strong bond to the tooth structure together with Adhese Universal



Cement

Reliable and versatile adhesive cementation with Variolink Esthetic



Protect

Protect restorations with the unique combination of active ingredients contained in Cervitec Plus