PrograPrint Clean®

Short Instructions
Kurzanleitung
Guide d'initiation rapide
Istruzioni d'uso brevi
Guía breve
Instruções de Uso Resumidas

Cleaning

PrograPrint Clean®

Table of Contents | Inhaltsverzeichnis | Table de matières | Indice | Índice | Lista de Conteúdo

 en	Short Instructions	4
 de	Kurzanleitung	14
 fr	Guide d'initiation rapide	24
 it	Istruzioni d'uso brevi	36
 es	Guía breve	48
 pt	Instruções de Uso Resumidas	58
	Instructions Anleitungen Instructions Istruzioni Instrucciones Instruções	68

English

1. About this document

These Short Instructions are part of the overall documentation for the device. They are only valid in combination with the Operating Instructions of the device ("Documents of equal applicability and required files", page 6).

WARNING! Improper use of the device.

Risk to the patient and the user.

- ▶ The Short Instructions contain important information on the safe handling of the device: Operating Instructions and Short Instructions must be read thoroughly prior to installation and operation.

These Short Instructions will help you to quickly familiarize yourself with the use of PrograPrint Clean.

Should you lose these Short Instructions, you can download them from www.ivoclar.com.

1.1. Signs and symbols

1.1.1 Warnings and additional information

Warnings

Warnings are used in these Short Instructions to warn you of a risk of injury to persons and/or damage to property.

- Please always read and observe these warnings.
- Follow all measures marked with the warning symbol and warning word.

Depending on the severity and the probability of the danger, the following warning levels are distinguished:

Warning symbol	Warning word	Danger level	Consequences resulting from non-observance
	DANGER	Immediate imminent danger	Death, severe personal injury
	WARNING	Potential danger	Death, severe personal injury
	CAUTION	Potential danger	Slight personal injury
	NOTICE	Potential danger	Damage to property

Explanation of the structure of a warning:

WARNING WORD! Name of source of danger, cause of hazard or type of risk.

Consequences of non-compliance with instructions.

- ▶ Action in order to avoid danger.

Example of a warning:

WARNING! Improper cleaning and maintenance of the device. Failure to observe the specified cleaning intervals defined in these instructions.

Risk to the patient and the user.

Damage to the device.

- ▶ Only carry out work on the device that is described in this chapter and assigned to the user.
- ▶ Do not use any cleaning agents for components, for which no explicit cleaning agents are mentioned.
- ▶ Always carry out the cleaning tasks at the specified intervals.

Additional information

Symbol	Meaning
	Additional information, e.g. for better understanding, for simplifying workflows or for further information.

1.1.2 Signs and symbols in the instructions

Symbol	Meaning
	The speed to be set on the magnetic stirrer
	The cleaning duration to be set on the magnetic stirrer

1.2. Warning symbols and mandatory signs on the device

Warning symbols and mandatory signs are attached to the device in order to warn you of damage to property and personal injury.

- Always observe the warning symbols and mandatory signs on the device.
- Follow the instructions described in the warning symbol or mandatory sign.

The following warning symbols are on the device:

Warning symbol	Meaning
	General warning sign
	Warning of inflammable substances
	Explosive atmosphere hazard warning

The following mandatory signs are on the device:

Mandatory sign	Meaning
	Observe the Operating Instructions
	Wear eye protection
	Wear protective gloves
	Work with extraction, use extraction in continuous operation

1.3. Documents of equal applicability and required files

For details on the documents of equal applicability, please contact your local trade partner or go to the download section of Ivoclar at www.ivoclar.com.

Document	Explanation
Operating Instructions PrograPrint Clean	Comprehensive information on the use of PrograPrint Clean
Instructions for Use for the different materials	Information about the use of the materials to be processed
Safety Data Sheet on the cleaning agent isopropanol (isopropyl alcohol)	Safety-relevant information on the cleaning agent isopropanol
Operating Instructions on the use of the 3D printer PrograPrint PR5	Information about requirements that must be observed for the transport, set-up and use of PrograPrint PR5
Operating Instructions for the post-curing unit PrograPrint Cure	Information about requirements that must be observed for the transport, set-up and use of PrograPrint Cure
Instructions for Use for PrograPrint CAM	Handbook for the PrograPrint CAM software
Short Instructions for: – PrograPrint PR5 – PrograPrint Clean (this document) – PrograPrint Cure	Excerpt of operation-relevant topics from the respective Operating Instructions

2. Safety

WARNING! Improper use of the device.

Hazard to the user.

► The safety chapter contains important information on the safety of the device: Please read it thoroughly prior to installation and operation.

The device is built according to state-of-the-art technology and recognized safety regulations. Nevertheless, operation bears the risk of injury to the user or third parties. Moreover, impairments of the device and other property are possible.

2.1. Intended use

PrograPrint Clean has been designed for cleaning the dental objects produced by the PrograPrint PR5.

Use the PrograMill Clean exclusively for this purpose.

In the case of damage caused by improper use or failure to observe the Operating Instructions, all liability and guarantee claims are void.

PrograPrint Clean is for cleaning with isopropanol that has the following properties:

- It must have a purity of at least 99.8%
- It must be suitable for cleaning medical products

Please note that only the materials and cleaning agents approved by the manufacturer ensure processing without damage to the machine.

All information regarding material and indication extensions corresponds to the state of knowledge at the time of going to print.

For additional details, go to <http://www.ivoclar.com>.

The Ivoclar Download Center is available at <https://downloadcenter.ivoclar.com/#lang=en>

The cleaning of non-approved 3D printed materials from other manufacturers is not recommended due to the lack of coordination with the printing/fabrication process.

2.1.1 Potential improper use

The device is not suitable for cleaning 3D printed objects from outside of the dental field. The following is considered as misuse of the system:

- Non-compliance with the specified intended use, intended operator specifications or intended environment
- Modifications, maintenance and repairs without prior authorization from Ivoclar
- Operation with inappropriate parameters
- Cleaning with non-approved cleaning materials

Improper use of the device can lead to the following:

- Danger for patients and operating personnel
- Danger of fire or explosion
- Impairment of the operability of the device
- Impairment of material properties and precision of printed objects

Ivoclar assumes no liability for damages resulting from improper use.

Typical misapplications of the device to be avoided:

Improper use	Consequences
Use of non-approved cleaning agents	<ul style="list-style-type: none"> – Contamination of dental objects and negative health effects in the patient – Danger of fire or explosion – Damage to the printed dental object
Incorrect cleaning and disposal	<ul style="list-style-type: none"> – Skin irritations and allergic reactions – Environmental damage – Damage to the device
Working with the device without protective equipment	Skin irritations and allergic reactions
Carrying out cleaning work on slippery floor	Danger of falling
Operating the device whilst the lid is open	<ul style="list-style-type: none"> – Evaporation of the cleaning agent – Leaking of flammable vapours
Blocking the self-closing lid with objects	<ul style="list-style-type: none"> – Skin irritations and allergic reactions – Danger of fire or explosion

Improper use	Consequences
Leaking or spilling of cleaning material due to the use of unsuitable canisters when refilling cleaning agent	<ul style="list-style-type: none"> – Skin irritations and allergic reactions – Danger of fire or explosion – Environmental damage – Damage to the device
Transportation of the device with full cleaning containers	<ul style="list-style-type: none"> – Leaking or spilling of the cleaning agent – Skin irritations and allergic reactions
Removing the cleaning container when it is full	<ul style="list-style-type: none"> – Danger of fire or explosion – Environmental damage
Filling the cleaning containers beyond the maximum marked level	<ul style="list-style-type: none"> – Damage to the device
Operating the device without using a fume hood / extraction cabinet in continuous operation (24 hours a day, 7 days a week)	<ul style="list-style-type: none"> – Evaporation of the cleaning agent – Leaking of flammable vapours – Skin irritations and allergic reactions – Danger of fire or explosion

2.1.2 Intended user

The device may only be operated by dental technicians and qualified dental staff for the fabrication of dental objects for dental applications.

The user is responsible for selecting the correct device settings.

2.1.3 Intended environment

The device is only permitted for use in closed rooms that are ventilated as specified, within the specified ambient conditions (see Technical data, section "Ambient conditions for storage and transportation without cleaning fluid" in the Operating Instructions, "Documents of equal applicability and required files", page 6) and in compliance with the specifications for safe installation (see requirements regarding the location site in the Operating Instructions, "Documents of equal applicability and required files", page 6).

2.2. Operator's obligations

The operator is responsible for the safe operation of the device.

- ▶ Ensure compliance and control:
 - Intended use
 - Statutory or other safety and accident prevention regulations
- ▶ Only operate the device in a technically perfect condition, in a proper, safety-conscious and risk-conscious manner and while observing the Operating Instructions ("Documents of equal applicability and required files", page 6).
- ▶ Keep these instructions and all documents of equal applicability complete, legible and accessible to personnel at all times.

2.3. Personnel qualifications

- ▶ Ensure that the personnel assigned to work with the device have read and understood these instructions and all documents of equal applicability, in particular safety, maintenance and repair information, before starting work.
- ▶ Ensure that personnel is aware of hazards and safety equipment ("Working areas, potential hazards and safety measures on the device", page 9).
- ▶ Manage the responsibilities, competence and monitoring of staff. All work must only be carried out by qualified technical staff.
- ▶ Personnel to be trained should only be working with the device under the supervision of qualified technical staff.

2.4. Staff obligations

- ▶ Only operate the device in a technically perfect condition, in a proper, safety-conscious and risk-conscious manner and while observing the Operating Instructions ("Documents of equal applicability and required files", page 6).
- ▶ Refrain from any process that could endanger personnel or third parties.
- ▶ In the case of safety-related malfunctions, switch off the device immediately and allow the fault to be rectified by the person responsible.
- ▶ In all cases of doubt regarding the safety of the device, switch off the device and prevent further use.
- ▶ In addition to the overall documentation, legal or other safety and accident prevention regulations including the applicable standards and guidelines of the country in which the device is operated must be complied with.
- ▶ Do not wear jewellery such as rings, bracelets or watches when working on the device to prevent cutting injuries or bruising.

2.5. Personal protective equipment

- ▶ For the protection against material and cleaning agent vapours or other dental material particles, wear a protective mask (half mask with protection class FFP3), gloves and protective gear suitable for the respective material (see Instructions for Use of the materials being used, "Documents of equal applicability and required files", page 6).

2.6. Working areas, potential hazards and safety measures on the device

As an operator, you are working at the areas described below.

The device is built according to state-of-the-art technology and recognized technical safety regulations. Hazardous areas, which cannot be avoided by design, are equipped with appropriate protective devices. Nevertheless, if the machine is used improperly, there is a danger to life or risk of injury to the user or third parties. In addition, the devices or dental applications may become damaged.

The working areas, their potential hazards and applicable safety measures are shown below:

CAUTION! Evaporation of cleaning agent due to the lid having been open for too long.

Skin irritations and allergic reactions.

Danger of fire or explosion.

- ▶ Do not block and keep open the self-closing lid with objects.

CAUTION! Evaporation of cleaning agent due to damaged seals around the lid and the cleaning container.

Skin irritations and allergic reactions.

Danger of fire or explosion.

- ▶ Check seals for damage before using the device.
- ▶ Do not operate the device if the seals are damaged.
- ▶ Do not remove the seals.

No.	Working area	Tasks / Work	Type of hazard	Safety equipment	Protective effect
1	Self-closing lid	Access to the cleaning containers	<ul style="list-style-type: none"> - Evaporation of the cleaning agent - Leaking of flammable vapours 	Self-closing lid	<ul style="list-style-type: none"> - Protection against skin irritation, allergic reactions - Protection against fire or explosion hazard - Protection against environmental damage
2	Cleaning containers	<ul style="list-style-type: none"> - Inserting the build platform - Cleaning dental objects - Changing and emptying the cleaning containers 	Leaking, slopping over material due to the build platform falling down into the full cleaning container	Holder for the build platform	Safe transfer of the build platform from one container to the other
			Spilled, slopping over material during filling or emptying the cleaning containers	Empty original containers for the disposal of cleaning agent	Safe disposal of the cleaning agent
			Inadequate printing result due to mix-up of containers for rough and fine cleaning	Signs to identify the containers	Safe printing results
			Leakage of cleaning liquid and flammable vapours	Seals between cleaning containers and base (housing)	<ul style="list-style-type: none"> - Protection against skin irritation, allergic reactions - Protection against fire or explosion hazard - Protection against environmental damage

2.6.1 Fume hood / extraction cabinet at the site of use

CAUTION! Evaporation of cleaning agent during use without fume hood / extraction cabinet.

Respiratory disorders, skin irritations and allergic reactions.
Danger of fire or explosion.

- ▶ Place and operate the device filled with isopropanol exclusively under an explosion-proof fume hood / in an explosion-proof extraction cabinet.
- ▶ Ensure that the explosion-proof fume hood / explosion-proof extraction cabinet is equipped for a minimum protection level of Zone 2 according to ATEX 137 (Definition Zone 2: A hazardous area classified as an atmosphere where a mixture of air and flammable substances in the form of gas, vapour or mist is not likely to occur in normal operation, but if it does occur, will persist for a short period only).
- ▶ Use the fume hood / extraction cabinet in continuous operation (24 hours a day, 7 days a week) if the cleaning containers are filled with isopropanol.
- ▶ Always comply with any additional country-specific standards / stipulations / regulations etc. on the usage / storage / disposal of isopropanol.

The fume hood / extraction cabinet must meet the following requirements

- Explosion-proof equipment
- Suitable for continuous operation (24 hours a day, 7 days a week)
- ATEX approved for each country of use
- No integrated plug socket, as the magnetic stirrer features a potted cable that must be led through the fume hood towards the outside.
- Measurements (L x W x H): 450 mm x 380 mm x 700 mm
- Air flow: The country-specific standards / regulations / specifications must be complied with.

2.7. Safety instructions for potential hazard areas

2.7.1 Magnetism

The inductive drive of the magnetic stirrer works with magnetism.

- ▶ Keep pacemakers, data carriers, magnetic cards and other devices that may be affected by magnetic fields away from the device.

2.7.2 Electrical safety

If covers are opened or parts are removed which are only accessible with tools, voltage-carrying components may be exposed. The plugs may also be under voltage. There is a risk of electric shock!

- ▶ Electrical work must only be carried out by your authorized service partner.
- ▶ Before connecting the device, check that the supply voltage and frequency are correct at the point of installation (see electrical data and the information on the type plate in the Operating Instructions, "Documents of equal applicability and required files", page 6).
- ▶ To avoid injury to the user and for cooling reasons and fire protection, never remove the housing covers.
- ▶ For installation, observe the requirements of the manufacturer for house installation.
- ▶ To prevent the risk of electric shock, connect the device only to a power supply with a protective conductor.
- ▶ The mains plug is used as disconnect device. Always connect the mains plug to an easily accessible protective contact socket.
- ▶ Connect the grounded and freely accessible protective contact socket to a separately secured circuit.
- ▶ Make sure that the protective contact socket is equipped with a residual current device (RCD).
- ▶ Connect the device to a separately secured circuit or make sure that no devices are connected that cause severe supply voltage fluctuations when switched on. These fluctuations interfere with the electronic controls and may cause failure of the system.

- ▶ Disconnect power before cleaning and maintenance work approved to be performed by the user.
- ▶ If you need to disconnect the unit from the power supply at a later time: Disconnect the plug from the mains socket, not on the device.

2.7.3 Hazardous materials

Cleaning agent (isopropanol)

Without the corresponding safety precautions, there is a risk of skin irritations, allergic reactions or respiratory diseases depending on the cleaning agent.

Due to the cleaning agent vapours, there is a risk of fire or explosion if the device and cleaning agents are handled incorrectly.

When working with cleaning agents, observe the following instructions:

- ▶ Observe the cleaning agent safety data sheet ("Documents of equal applicability and required files", page 6).
- ▶ Store, use and dispose of cleaning products in accordance with the applicable national regulations and safety data sheets.
- ▶ Observe applicable fire regulations when handling the cleaning agent.
- ▶ Wear protective gear ("Personal protective equipment", page 8).
- ▶ To avoid a fire or explosion hazard, do not spill the cleaning agent.
- ▶ Observe the requirements for the installation site (see specifications for the position in the Operating Instructions, "Documents of equal applicability and required files", page 6).
- ▶ In order to avoid a fire or explosion hazard due to a faulty device or defective safety features, operate the device only in perfect technical condition, in particular with fitted, intact seals.

2.8. Safety information for the individual operating phases

2.8.1 Transport

- ▶ Transport the device only as described in the Operating Instructions (see requirements regarding transport and setting up in the Operating Instructions, "Documents of equal applicability and required files", page 6).
- ▶ In order to avoid corrosion on the device and subsequent damage to the device, transport and store device only in the permissible temperature range and ambient conditions (see requirements regarding the ambient conditions in the Operating Instructions, "Documents of equal applicability and required files", page 6).

2.8.2 Initial operation

- ▶ Ensure that this device is only operated by authorized and trained specialists.
- ▶ If the device has been stored in a cold environment or at high humidity, maintain a drying or temperature adjustment time of approx. 4 hours (without voltage) at room temperature before initial operation.
- ▶ Before connecting the device, check that the supply voltage and frequency are correct at the point of installation (see electrical data in the Operating Instructions, "Documents of equal applicability and required files", page 6).
- ▶ The requirements regarding the location site and ambient conditions must be observed (see requirements regarding the location site and ambient conditions in the Operating Instructions, "Documents of equal applicability and required files", page 6).

2.8.3 Operation

- ▶ Ensure that this device is only operated by authorized and trained specialists.
- ▶ Keep unauthorized persons, such as patients and children, as well as animals away from the unit.
- ▶ In all cases of doubt regarding the safety of the device, switch off and disconnect the device and take suitable measures to prevent further use.
- ▶ Prior to connecting power or operation, check the device, the accessories and protective equipment for any damage. Do not use damaged, non-functioning equipment or accessories; instead notify your authorized service partner.

- ▶ In order to ensure the product safety and warranty services, the device must be exclusively operated with the original accessories from Ivoclar, particularly the original power cord. The user bears the risk when using non-approved accessories.
- ▶ Ivoclar recommends using only the approved cleaning agent to ensure process reliability.
- ▶ Never bypass the safety equipment of the device or set it out of operation ("Working areas, potential hazards and safety measures on the device", page 9).
- ▶ To prevent damage to the device and a reduction of the device performance, observe the cleaning requirements and cycles (see "Cleaning by the user", page 13 in these Short Instructions as well as the overview and instructions for the cleaning and maintenance work in the Operating Instructions, "Documents of equal applicability and required files", page 6).
- ▶ Only operate the device unsupervised if the operating conditions for unsupervised operation described below are fulfilled.

Unsupervised operation:

The device may be operated unsupervised, provided the national and local laws and provisions allow for such action and provided that they are observed. Furthermore, the requirements of the respective insurance company must be met.

- ▶ Never use the device if the cleaning containers are heavily soiled.
- ▶ Protect the device against unauthorized access.

2.8.4 Cleaning by the user

- ▶ Only clean as specified in the Operating Instructions and observe the related safety regulations (see overview and requirements regarding cleaning and maintenance work in the Operating Instructions, "Documents of equal applicability and required files", page 6).

2.8.5 Improper maintenance, modifications and repairs

Improper service and improper repairs or modifications will endanger patients and users and result in damage to the device and the final products.

Should you carry out any repairs or maintenance of the device or make modifications or remove the housing without prior written consent by an authorized service partner, all warranty claims are void.

Unauthorized opening and removal of components can expose voltage-carrying components. The plugs may also be under voltage. There is a risk of electric shock!

- ▶ Maintenance as well as repairs of damaged safety equipment or machine parts must only be carried out by a service partner authorized by Ivoclar.

3. Additional information

- Download link Operating Instructions PrograPrint System:
<https://www.ivoclar.com/prograprint-manual>

The PrograPrint Clean device filled with isopropanol may only be used with a fume hood in continuous operation. For more information, refer to the PrograPrint Clean Operating Instructions under:
<https://www.ivoclar.com/prograprint-manual>

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Ivoclar Vivadent Inc.
Ivoclar Vivadent AG