


Variolink Esthetic® DC


Placement of inlays (glass-ceramic, e.g. IPS e.max®)

Try-in


Variolink
Esthetic
Try-In

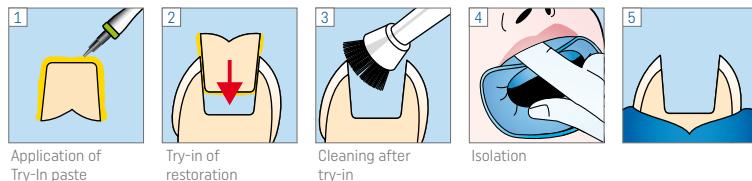



Restoration

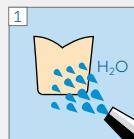



Monobond
Etch & Prime

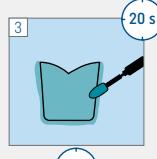


Preparation

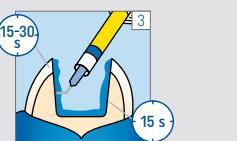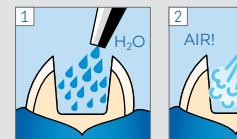

Total Etch:
optional

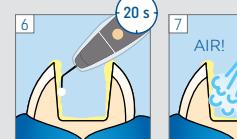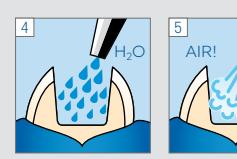


Adhere
Universal

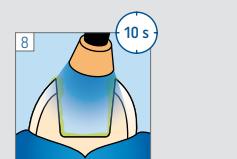

Bluephase
G4

Cementation


Variolink
Esthetic DC

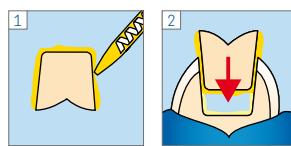

Bluephase
G4




Liquid
Strip

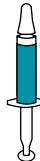

Polishing

Fluor
Protector S

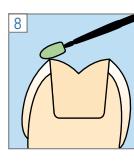

Manufacturer
Ivoclar Vivadent AG
Bendererstrasse 2
9494 Schaan/Liechtenstein
www.ivoclar.com



Date information prepared:
2022-08-23
681581/DE/Rev.1



more information
www.cementation-navigation.com

ivoclar

Variolink Esthetic® DC

Placement of crowns and bridges (glass-ceramic, e.g. IPS e.max®)


Einprobe



Restoration



Monobond Etch & Prime

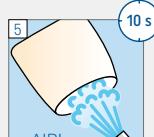


Preparation




Total Etch:

optional


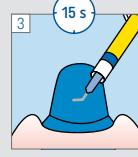


Adhere Universal

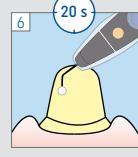

Avoid pooling


Cementation

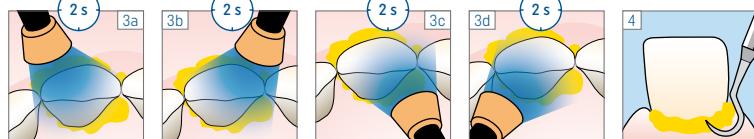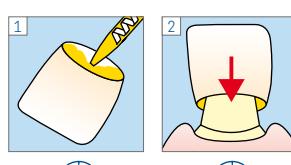

Secure the restoration until final polymerisation



* per segment and mm ceramic

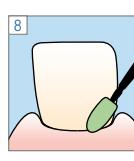

Manufacturer
Ivoclar Vivadent AG
Benderstrasse 2
9494 Schaan/Liechtenstein
www.ivoclar.com



Date information prepared:
2022-08-23
681581/DE/Rev.1



more information
www.cementation-navigation.com

ivoclar