


Variolink® Esthetic LC


Placement of Veneers (glass ceramic, composite)

Try-in


Variolink Esthetic Try-in

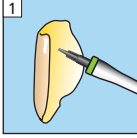


Application of Try-in paste

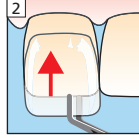


Try-in of restoration

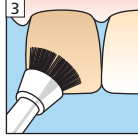


Cleaning after Try-in

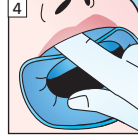


Isolation

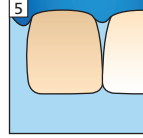

Restoration

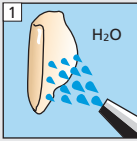

H₂O

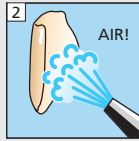


AIR!


Pre-treatment of ceramic restorations:
According to manufacturer's instructions
E.g. IPS Empress: Etch for 60 s
IPS e.max CAD/Press: Etch for 20 s
with IPS Ceramic etching gel (hydrofluoric acid). Follow safety instructions.

Pre-treatment of composite restorations:
According to manufacturer's instructions
E.g. SR Nexco, 2 bar, Al₂O₃ 100 µm
or roughened with finishing diamonds.

Monobond Plus

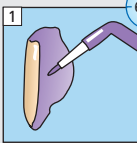


60 s

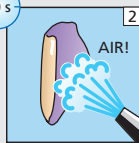

AIR!

Total Etch optional


Preparation

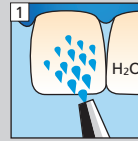


H₂O



AIR!

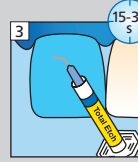

15-30 s

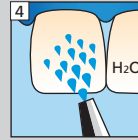


H₂O




AIR!



Adhese Universal

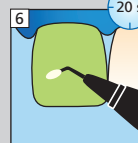

20 s

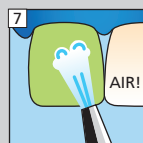


AIR!

Avoid pooling





Bluephase Style

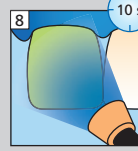


10 s

Cementation


Variolink Esthetic LC

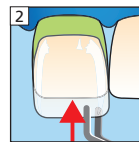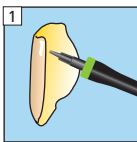

Bluephase Style


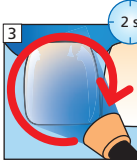


2 s

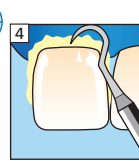

Secure the restoration until final polymerisation


Liquid Strip

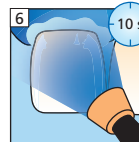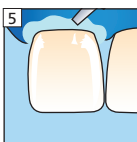

10 s

Polishing

Fluor Protector

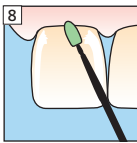

more information
www.cementation-navigation.com

CE 0123



Please read
Instruction for Use

ivoclar
vivadent
clinical


Date information prepared: 2014-06-25
666332/e/Rev. 0

Variolink® Esthetic LC


Placement of inlays (thickness < 2 mm) (glass ceramic, composite)

Try-in


Variolink Esthetic Try-in

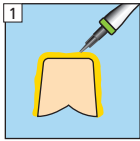


Application of Try-in paste

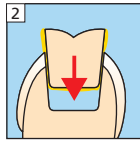


Try-in of restoration

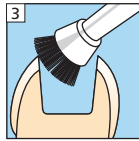


Cleaning after Try-in

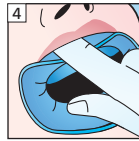

Isolation

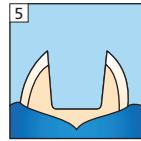


Restoration

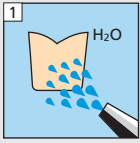

Pre-treatment of ceramic restorations:
According to manufacturer's instructions
E.g. IPS Empress: Etch for 60 s
IPS e.max CAD/Press: Etch for 20 s
with IPS Ceramic etching gel (hydrofluoric acid). Follow safety instructions.






Pre-treatment of composite restorations:
According to manufacturer's instructions
E.g. SR Nexco, 2 bar, Al₂O₃ 100 µm
or roughened with finishing diamonds.

Monobond Plus

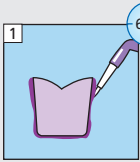

Total Etch optional


Preparation

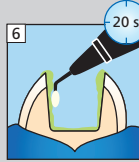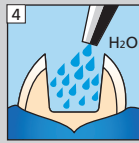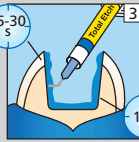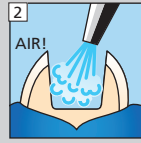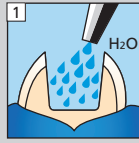

Avoid pooling





Bluephase Style


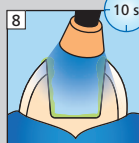


Cementation

Variolink Esthetic LC


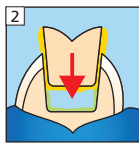
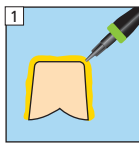


Bluephase Style

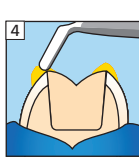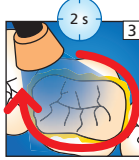

Secure the restoration until final polymerisation


Liquid Strip

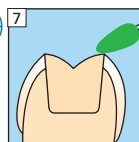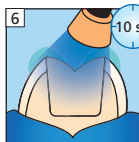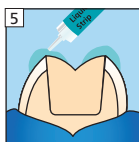

* per segment and mm ceramic

Polishing

Fluor Protector

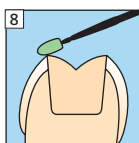

more information
www.cementation-navigation.com

CE 0123



Please read
Instruction for Use

ivoclar
vivadent
clinical