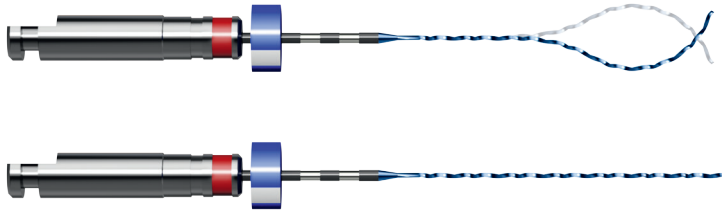
# XP-endo Finisher

Protocol  
Protocole  
Protokoll

STERILE R

## ► Golden rules

800 rpm  
1 Ncm

- XP-endo Finisher should be used only after canal preparation to at least #25;
- In multirooted teeth, start with the largest canal;
- The access cavity should be filled with irrigant only after the XP-endo Finisher is in the canal.

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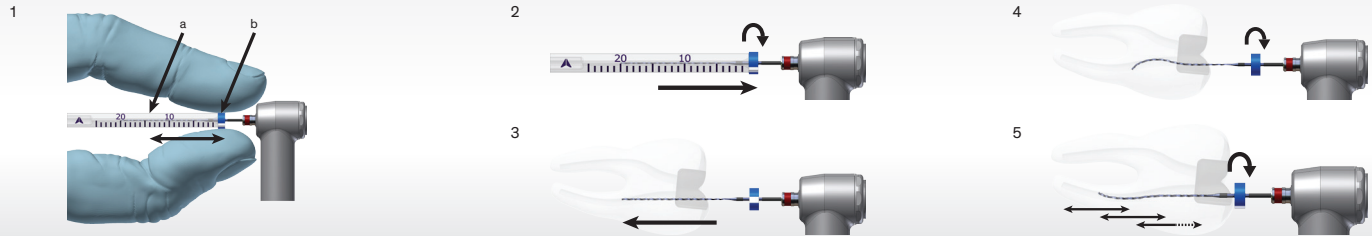
## ► Protocol

800 rpm (800-1000)  
1 Ncm

- Fix the canal working length by using the plastic tube (a) to adjust the rubber stop (b) (Figure 1).
- Cool the XP-endo Finisher down inside the tube using a cold spray.
- Put the XP-endo Finisher in rotation mode and remove it from the tube by applying a lateral movement to ensure the XP-endo Finisher remains straight (Figure 2). Turn off the rotation.
- Insert the XP-endo Finisher into the first canal of the tooth while straight (Figure 3). Once the tip is inside, turn on the rotation and insert file (Figure 4). Add irrigant to the access cavity.
- Use the XP-endo Finisher for approximately one (1) minute, using slow and gentle 7-8 mm lengthwise movements to contact the full length of the canal (Figure 5). Make parietal movements during the procedure. Be careful to stay in the canal.
- After one minute, remove the XP-endo Finisher from the canal while it's still in rotation.

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