All-Ceramics

Preparation

Guidelines

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Preparations as well as the images are by courtesy of Prof. Dr D Edelhoff/Germany

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How do you achieve the perfect preparation design for all-ceramic restorations?

These guidelines on the preparation of teeth to be restored with either IPS e.max or IPS Empress all-ceramic material provide complementary information to that contained in the All-Ceramic Preparation Guide, Edition II.

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**Marginal
orientation groove**

Marginale Orientierungsrille

**Central and incisal
orientation grooves**

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**Facial preparation
involving 3 levels**

Faziale Präparation in
3 Ebenen

**Initial proximal
separation**

Initiale proximale
Separation

**Final proximal
preparation**

Definitive proximale
Präparation

**Finishing and smoothing
of the preparation**

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Pre-operative situation

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**Incisal
orientation grooves**

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**Facial and oral aspect:
marginal orientation groove**

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**Facial aspect: central and
incisal orientation grooves**

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Facial preparation involving 3 levels; oral concavity

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Finishing and smoothing of the preparation

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Partial crown | *Teilkrone*

Pre-operative situation

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**Cavity preparation,
occlusal orientation grooves**

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**Occlusal and oral reduction
of the cusp**

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**Creation of a circular
preparation margin**

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Proximal preparation

Approximale Präparation

**Finished and smoothed
preparation**

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**Preparation of
the proximal box**

Approximal:
Präparation Kasten

**Smoothing of the cavity
and the proximal boxes**

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**Preparation after finishing
and smoothing**

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**Reduction of the
occlusal surface**

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Creation of a circular preparation margin

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Following cavity preparation, the shade of the prepared tooth is determined with the IPS Natural Die Material shade guide. This enables the correct materials (as regards shade, opacity and translucency) to be chosen for the all-ceramic restoration and the correct shade and brightness value to be selected.

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Ivoclar Vivadent AG
Ivoclar Vivadent GmbH