SR Ivolen®

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For dental use only.

Description

- Custom impression trays
- Customized base plates for:
 - wax rims
 - intraoral registration devices
 - the tooth set-up with wax
 - the fabrication of individualized anterior guidance plates (e.g. in Stratos®)

Technical data

Colour: yellow

Indications

Suitable for use in the treatment of partially or completely edentulous patients

Contraindications

The use of SR Ivolen is contraindicated if the patient is known to be allergic to any of its ingredients.

Limitations of use

- The material must not be processed in the oral cavity.
- The material is unsuitable for alginates and hydrocolloids.
- The product is not intended to be reprocessed or reused.

Side effects

There are no known side effects to date.

Composition

- Powder: Polymethacrylate copolymer, poly(methylmethacrylate), calcium carbonate, poly(ethylmethacrylate), contains dibenzoyl peroxide
- Liquid: Methyl methacrylate, paraffin oil

Application

1. After blocking out the undercuts, soak the model in water and isolate it with Ivoclar Vivadent Separating Fluid.
2. Measure out the powder and liquid according to the instructions: 20 g of SR Ivolen powder, 6 ml of SR Ivolen liquid. Mix well in the mixing cup using a spatula until the mixture no longer sticks to the mixing cup. Knead by hand for a few seconds.
3. SR Ivolen can be immediately applied and shaped; the material is not sticky. Alternatively, the material can be pressed between two polyethylene sheets with a compression roller to form a layer of uniform thickness.
4. Adapt SR Ivolen to the cast; shape with a suitable instrument and cut off excess.
5. You may use excess to form tray handles.
6. After the material has self-cured for approx. 7 minutes, trim the margins with burs or similar instruments. If desired, you may also polish SR Ivolen.

Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: www.ivoclarvivadent.com, and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website (www.ivoclarvivadent.com).

Warnings

- The monomer contains methyl methacrylate (MMA).
- MMA is irritating and easily flammable (flash point: +10 °C/50 °F).
- MMA is irritating to the eyes, skin and respiratory system.
- May cause sensitization upon skin contact
- Avoid contact of the skin with monomer or uncured material. Commercially available medical gloves do not offer protection against the sensitizing effect of methacrylates.
- Do not inhale vapours.
- Keep away from sources of ignition. Do not smoke.
- Do not empty into drains.
- Take precautionary measures against electrostatic charge.
- Do not inhale grinding dust during finishing – use suction equipment and a face mask.

Disposal information

- Remaining stocks must be disposed of according to the corresponding national legal requirements.

Shelf life and storage


- Storage temperature 2–28°C (36–82°F)
- Do not use the product after the indicated date of expiration.
- Expiration date: See note on packaging.

Additional information

Keep material out of the reach of children!

Image description:

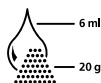Mixing ratio:
Powder 20 g
Liquid 6 ml

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions.

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Ivoclar Vivadent Limited
Ivoclar Vivadent, Inc.