**Description**

Multilink Automix try-in pastes are water-soluble glycerine pastes. The try-in pastes enable the simulation of the shade effect of all-ceramic restorations (inlays, onlays, crowns) that are to be cemented with Multilink Automix luting composite.

It is important to note that the shade and translucency of the try-in pastes corresponds to that of the **cured (!)** Multilink Automix luting composite.

**Shades**

Multilink Automix try-in pastes are available in four shades with different degrees of translucency:

- transparent (high translucency)
- yellow (high translucency)
- opaque (low translucency)
- white (medium translucency)

The shade of the try-in paste may subtly differ from that of the corresponding uncured Multilink Automix luting composite, since the luting composite attains its final shade only after it has been cured.

The shade effect of the try-in pastes is coordinated with that of the fully cured luting composite of the the same thickness.

**Composition**

The Multilink Automix try-in pastes contain glycerine, mineral fillers and dyes.

**Indication**

Simulation of the shade of the permanent luting composite.

**Contraindication**

Do not use if the patient is known to be allergic to any of the ingredients of the Multilink Automix try-in pastes.

**Side effects**

The hygroscopic try-in pastes cause fluid movement within the dentin tubules. As a result, the patient may feel temporary discomfort. Therefore, the try-in procedure should be carried out under anaesthetic or after desensitization with a desensitizer (e.g. Telio Desensitizer).

**Interactions**

Completely rinse any try-in pastes residue from the restoration and the tooth with water prior to applying the permanent cement. Otherwise, the bond between the restoration and Multilink Automix may be compromised.

**Application**

- Shade simulation with the help of try-in pastes must always be carried out before the working field is isolated or the teeth are dried, since drying temporarily lightens the tooth structure.
- Apply a small amount of the selected try-in paste to the restoration and carefully seat it without pressure. If the restoration cannot be seated easily, check the inner surface of the restoration as well as the proximal regions for any premature contacts. Do not force the restoration into place (risk of fracture!).

**Note:**

Care should be taken when checking the occlusion of brittle ceramic restorations before they are permanently cemented, as there is a risk of fracture.

- The try-in pastes are water-soluble and can be easily removed under running water or with ultrasonic cleaners. The restoration should be thoroughly cleaned and dried.
- Thoroughly clean the tooth structure with water spray.
- Permanently seat the restoration with Multilink Automix (See Multilink Automix Instructions for Use).

**Storage**

- Do not use Multilink Automix try-in pastes after the expiry date.
- Store at 2-28 °C / 36-82 °F.
- Expiry date: See information on syringes and packaging

**Keep material out of children's reach.****For use in dentistry only.**

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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**Ivoclar Vivadent Marketing Ltd.**
**Ivoclar Vivadent AB**
**Ivoclar Vivadent Limited**
**Ivoclar Vivadent, Inc.**