Description

Helioseal® F Plus is a light-curing, white-shaded fissure sealant featuring fluoride release.

Composition

Dimethacrylates

Indication

Helioseal F Plus is used to seal fissures, pits and foramina caeca.

Contraindication

- If a patient is known to be allergic to any of the ingredients of Helioseal F Plus, it must not be used.
- The use of Helioseal F Plus is contraindicated if a dry working field cannot be established.

Side effects

In rare cases, components of Helioseal F Plus may lead to sensitization. The product must not be used in such cases.

Other interactions

None known to date.

Application**1. Clean the enamel surface**

Thoroughly clean the enamel surface to be sealed (e.g. Proxyl®).

2. Isolation

Isolate the working field, preferably with a rubber dam (e.g. OptraDam® Plus).

3. Conditioning

Conditioning is performed according to the Instructions for Use of the product in use. Ivoclar Vivadent recommends the use of a low-viscosity phosphoric acid gel.

4. Rinse thoroughly and dry

- Thoroughly rinse off the conditioner using water spray.
- Dry with water- and oil-free air.
- The etched enamel should have a mat white appearance; avoid contamination of the etched surface with saliva. In case of contamination, the conditioning must be repeated.

5. Application of Helioseal F Plus

- Apply Helioseal F Plus directly with the single-use cannula or the cavifil to the tooth surface, and disperse.
- Cure the sealant using a suitable curing light. For detailed information on the exposure time see Table 1.
- Check the seal and occlusion.

Special notes

1.  For single use only. If Helioseal F Plus is applied directly in the mouth of the patient, we recommend using the Cavifil or the tips of the syringe only for one patient due to hygienic reasons (prevention of cross-contamination between patients).
2. Do not disinfect syringes with oxidizing disinfectants.
3. In order to prevent air bubbles in the syringe, do not pull back the plunger of the syringe during or after the treatment.

Warning

- Avoid contact of unpolymerized Helioseal F Plus with skin/mucous membrane or eyes. Unpolymerized Helioseal F Plus may cause slight irritation and, in rare cases, may lead to sensitization against methacrylates.
- Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Shelf life and storage

- Storage temperature 2–28 °C (36–82 °F)
- Close the syringe immediately after use. Exposure to light may lead to premature polymerization.
- Expiration date: See note on Cavifil, syringe and/or packaging
- Do not use Helioseal F Plus after the indicated date of expiration.

Keep out of the reach of children!**For use in dentistry only!**

The product has been developed for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the product for its suitability and use for any purpose not explicitly stated in the Instructions.

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